Det intelligente drikkeglas

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06210801

Indholdsfortegnelse

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11 Disclaimer

This device is meant for personal uses only. This device is not intended for medical purpose, please refer to a physician or health professional. Your use of the device is entirely at your own risk, for which we shall not be liable. It shall be your own responsibility to ensure that any products, services, data or information available meet your specific requirements.

This device makes use of load cell principle and GSM wireless data transmission and is provided with a power adaptor included in the packaging. This device is only compatible with the provided power adaptor for 230V. Please pay careful attention to the labeling of the power adaptor for correct use.

This device is compatible with GSM at the frequencies of 790MHz-870MHz. The device will not work in North America.

This device allows direct access to drinking data on display on the device as well as remote access to cloud-based drinking data from the device. The measured data is only representing the amount of liquid being emptied from the vessel. This may not be representative for the amount being consumed by the user.

This device is restricted to one person use at a time. When drinking vessel is separated from the electronics (part with display), the drinking vessel can tolerate dishwashing between uses. If the device is transferred from one person to another person for usage, the device should be properly cleaned by competent health professional using 70% IPA (iso-propyl-alcohol) between use and the ID of the device linked to the new user to avoid any mix-up of drinking data. The manufacturer reserves the right not to be responsible or liable for the use of data generated from the device.

The device is covered by 2 year warranty, with the right to repair or exchange device based on manufacturers decision. Damage due to normal wear and tear or neglect is not covered by the warranty. New product, drinking vessel or power adaptor is available for order.

If personal data (email, name, addresses, social security number) from the user is associated/linked with data generated by the drinking vessel, the input of these personal data takes place voluntarily. The manufacturer has no access to personal data. Any personal data provided as input by user via the website access provided by the Manufacturer is encrypted.

The manufacturer assumes no liability when device or data is used integrated with software systems and platforms not supplied by the manufacturer.

12 AquaTime Device Safety and Software Update information

Safe Battery Handling and Usage

Battery can EXPLODE, CATCH FIRE and/or CAUSE BURNS if disassembled, punctured, cut, crushed, short circuited, incinerated, exposed to water, fire or high temperatures.

To ensure optimal safety and performance:

- **Only use AquaTime electronics and drinking vessel and accessories.** If you are unsure about whether a replacement charger/power adaptor is compatible, contact device manufacturer.
- **DO NOT** use the detachable part containing electronics for anything other than its intended purpose.
- **DO NOT** use the device if the charger/power adaptor, or the part with electronics or drinking vessel appears to be damaged or compromised. Examples includes, but are not limited to: leaking, cracks, melting, odors, corrosion and the electronics and drinking vessel being non-attachable
- **DO NOT** disassemble the electronics secured by screws, crush, short external contacts, or dispose of in fire or water.
- **DO NOT** expose the power adaptor and/or the detachable part containing electronics to fire or temperatures greater than 60 degrees Celcius. Do not place next to areas which may become hot. Examples includes, but are not limited to: on or near a space heater, cooking surface, cooking appliance, iron, radiator, or fire place.
- **DO NOT** get the power adaptor and/or the detachable part containing electronics wet. Even Though they might dry and appear to operate normally, the circuitry can slowly corrode and defeat its safety circuitry.
- **DO NOT** store the detachable part containing electronics with hazardous or combustible material. Store in a cool, dry, ventilated area.
- **ALWAYS** keep the power adaptor and detachable part containing electronics away from children.
- **MAKE SURE** the charger and power connection to the product have empty compartments and free from any moisture or loose debris before connecting the power.
- **AVOID** dropping the product. Dropping the product, especially on a hard surface, might cause damage to the power adaptor, drinking vessel and the electronics. If you suspect any damage to power adaptor, drinking vessel or electronics, please replace it.
- **FOLLOW** the USER GUIDE
- **FOLLOW** local guidelines for dispose of used electronics containing batteries.

To the extent permissible by applicable law, failure to follow these warnings and/or directions could result in damage to AquaTime product that will not be covered by AquaTime warranty.

13 Compliance with 2014/53/EU Radio Equipment Directive (RED)

AquaTime wireless products comply with the requirement of Article 10(2) of the RED as they can be operated in at least one Member State as examined. The product also complies with Article 10(10) as it has no restrictions on putting into service in all EU Member States.

This product is for indoor use only. In accordance with Article 10.8 (a) and 10.8 (b) of the RED, the following tables provide information on the frequency bands used and the maximum radio frequency transmit power of AquaTime wireless product for sale in the EU

Details of Operation:	SARA-N210	SARA-N200
Description of Service	E-UTRA Band 20	E-UTRA Band 8
Transmit frequency	832 MHz to 862 MHz	880 MHz to 915 MHz
Receive frequency	791 MHz to 821 MHz	925 MHz to 960 MHz
Modulation	BPSK, QPSK	BPSK, QPSK
Power Class	Class 3	Class 3
Transmit power	23.5 dBm. conducted	23.5 dBm. conducted

Maximum Antenna Gain	Maximum Antenna Gain (dB)	Power Density (mW/cm ²)
NB LTE 20	9.2	0.4157

13.1 Power Adapter Safety

Do not drop or cause impact to the power adaptor

If the power cable is damaged (for example, the chord is exposed or broken) or the plug loosens, stop using immediately. Continued use may led to electric shocks, short circuits, or fire.

Do not tough the power adaptor with wet hands. Doing so may lead to short circuits, malfunction, or electric shocks.

13.2 Accessory Safety

Using an unapproved or incompatible power adaptor, USB or battery may cause fire, explosion or other hazards.

Choose only accessories approved for use with this model by the device manufacturer. The use of any other types of accessories may void the warranty, may violate local regulations and laws, and may be dangerous.

Please contact your retailer for information about the availability of approved accessories in your area.

13.3 Device Safety

Maximum Permissible Exposure (MPE). Ensure that at least 31 cm separation distance is maintained between the AquaTime wireless device and the body of the User, when not drinking from the device.

The socket outlet should be installed near the equipment and easily accessible.

Do not plug device into Powerline pass-through sockets that exceed the product ratings.

Do not plug major home appliances into the filtered AC socket or into an attached power strip.

13.4 Specific Precautions for EMC

Warning: AquaTime class A equipment is compliant with Class A of EN 55032. In a residential environment this equipment may cause radio interference.

AquaTime product are tested to the requirements of EN 55032 (Emission) and EN 55024 (immunity). There are no specific precautions which must be taken in order to comply with the requirements of the Directive 2014/30/EU Essential Requirements in Section 1 of Annex I.

In addition a AquaTime wireless product is assessed/tested to ensure spurious emissions are within the specified limits, as well as meeting the requirement for adaptivity, which mitigate against problems caused by co-location with other wireless products.

AquaTime products are not subject to the requirements in Section 2 of Annex I of the Directive 2014/30/EU (for fixed installation).

13.5 Disposal of Waste Batteries and Electrical Equipment

This Symbol on the product or its packaging means that this product and any batteries it contains must not be disposed of with your household waste. Instead, it is your responsibility to hand over to an applicable collection point for the recycling of batteries and electrical equipment.

This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical equipment, which could be caused by inappropriate disposal. Disposal regulations vary by region. Dispose of old batteries and equipment in compliance with your local regulations.

13.6 Software Update

Software updates can be released by AquaTime to fix bugs or enhance functions after the product has been released. Updated software versions released will not impact continued compliance with the applicable rules. Radio Frequency Parameters are not accessible for the User and cannot be changed by the user.

13.7 Compliance

For EU Declaration of Conformity, visit www.aquatime.dk/help/CE

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